

Patient Table Field Names

4DM VARIABLE NAME	DEFAULT FIELD IN TABLE	DATA TYPE	SIZE
PAT_NAME	PAT_NAME*	SQL_CHAR	32
PAT_ID*	PAT_ID*	SQL_CHAR	24
STUDY_INSTANCE_UID	STUDY_INSTANCE_UID*	SQL_DATE	12
STUDY_DATE	STUDY_DATE	SQL_DATE	12
STUDY_TIME	STUDY_TIME	SQL_TIME	12
STR_STUDY_DATE	STR_STUDY_DATE	SQL_DATE	12
STR_STUDY_TIME	STR_STUDY_TIME	SQL_TIME	12
RST_STUDY_DATE	RST_STUDY_DATE	SQL_DATE	12
RST_STUDY_TIME	RST_STUDY_TIME	SQL_TIME	12
PAT_BDATE	PAT_BDATE	SQL_DATE	12
PAT_AGE	PAT_AGE	SQL_INTEGER	2
PAT_SEX	PAT_SEX	SQL_CHAR	16
PAT_WGT	PAT_WGT	SQL_INTEGER	2
PAT_HGT	PAT_HGT	SQL_FLOAT	4
PAT_BSA	PAT_BSA	SQL_FLOAT	4
WGT_UNITS	WGT_UNITS	SQL_CHAR	40
HGT_UNITS	HGT_UNITS	SQL_CHAR	40
BSA_UNITS	BSA_UNITS	SQL_CHAR	40

ACCESSION_NUMBER	ACCESSION_NUMBER	SQL_CHAR	128
PROCEDURE_ID	PROCEDURE_ID	SQL_CHAR	64
SCHED_STEP_ID	SCHED_STEP_ID	SQL_CHAR	64
ORD_PHYSICIAN	ORD_PHYSICIAN	SQL_CHAR	32
REF_PHYSICIAN	REF_PHYSICIAN	SQL_CHAR	32

REP_PHYSICIAN	REP_PHYSICIAN	SQL_CHAR	32
DATA_STATE	DATA_STATE	SQL_CHAR	64
EXPORT_TIME	EXPORT_TIME	SQL_CHAR	24

STUDY_INDICATIONS	STUDY_INDICATIONS	SQL_CHAR	1024
CHEST_PAIN_HISTORY	CHEST_PAIN_HISTORY	SQL_CHAR	80
CHEST_PAIN_CLASS	CHEST_PAIN_CLASS	SQL_CHAR	40
PRE_TEST_LIKELIHOOD	PRE_TEST_LIKELIHOOD	SQL_CHAR	40

STRESS_PROTOCOL	STRESS_PROTOCOL	SQL_CHAR	80
STRESS_PROTOCOL_RESULTS	STRESS_PROTOCOL_RESULTS	SQL_CHAR	1024
STRESS_ADEQUACY	STRESS_ADEQUACY	SQL_CHAR	40
STRESS_TEST_SUMMARY	STRESS_TEST_SUMMARY	SQL_CHAR	1024
STRESS_CHEST_PAIN_SYMPTOMS	STRESS_CHEST_PAIN_SYMPTOMS	SQL_CHAR	40
STRESS_SYMPTOMS	STRESS_SYMPTOMS	SQL_CHAR	80
REST_CHEST_PAIN_SYMPTOMS	REST_CHEST_PAIN_SYMPTOMS	SQL_CHAR	40
EXERCISE_DURATION	EXERCISE_DURATION	SQL_FLOAT	4

Field names with * are keyed fields in table.

4DM VARIABLE NAME	DEFAULT FIELD IN TABLE	DATA TYPE	SIZE
EXERCISE_MAXSTAGE	EXERCISE_MAXSTAGE	SQL_CHAR	40
EXERCISE_MAXSTAGE_DURATION	EXERCISE_MAXSTAGE_DURATION	SQL_CHAR	16
EXERCISE_METS	EXERCISE_METS	SQL_FLOAT	4
EXERCISE_FUNCTIONAL_CLASS	EXERCISE_FUNCTIONAL_CLASS	SQL_CHAR	40
EXERCISE_REASON_FOR_TERM	EXERCISE_REASON_FOR_TERM	SQL_CHAR	80
PHARMACOLOGIC_INFUSION_RATE	PHARMACOLOGIC_INFUSION_RATE	SQL_INTEGER	2
PHARMACOLOGIC_INFUSION_DUR	PHARMACOLOGIC_INFUSION_DUR	SQL_INTEGER	2
VASO_EXER_PROTOCOL	VASO_EXER_PROTOCOL	SQL_CHAR	1024

EKG_RESULTS	EKG_RESULTS	SQL_CHAR	1024
STRESS_EKG_STATUS	STRESS_EKG_STATUS	SQL_CHAR	40
STRESS_EKG_RHYTHMS	STRESS_EKG_RHYTHMS	SQL_CHAR	40
STRESS_EKG_AVBLOCKS	STRESS_EKG_AVBLOCKS	SQL_CHAR	40
STRESS_EKG_INTRACONDUCTIONS	STRESS_EKG_INTRACONDUCTIONS	SQL_CHAR	40
ST_RESPONSE	ST_RESPONSE	SQL_CHAR	128
STRESS_ARRHYTHMIAS	STRESS_ARRHYTHMIAS	SQL_CHAR	256
REST_EKG_STATUS	REST_EKG_STATUS	SQL_CHAR	40
REST_EKG_RHYTHMS	REST_EKG_RHYTHMS	SQL_CHAR	40
REST_EKG_AVBLOCKS	REST_EKG_AVBLOCKS	SQL_CHAR	40
REST_EKG_INTRACONDUCTIONS	REST_EKG_INTRACONDUCTIONS	SQL_CHAR	40
REST_EKG_REPOLARIZATIONS	REST_EKG_REPOLARIZATIONS	SQL_CHAR	40
REST_ARRHYTHMIAS	REST_ARRHYTHMIAS	SQL_CHAR	256

STRESS_HR	STRESS_HR	SQL_INTEGER	2
REST_HR	REST_HR	SQL_INTEGER	2
STRESS_BP	STRESS_BP	SQL_CHAR	8
REST_BP	REST_BP	SQL_CHAR	8
EXERCISE_PRP	EXERCISE_PRP	SQL_INTEGER	2
HR_CHANGE	HR_CHANGE	SQL_CHAR	40
HR_AGE_PERCENT_MAX	HR_AGE_PERCENT_MAX	SQL_INTEGER	2
BP_CHANGE	BP_CHANGE	SQL_CHAR	40
BP_RESPONSE	BP_RESPONSE	SQL_CHAR	40

IMAGING_PROTOCOL	IMAGING_PROTOCOL	SQL_CHAR	64
STUDY_PROTOCOL_DATA	STUDY_PROTOCOL_DATA	SQL_CHAR	1024
STRESS_RADIOPHARMACEUTICAL	STRESS_RADIOPHARMACEUTICAL	SQL_CHAR	18
STRESS_DOSE	STRESS_DOSE	SQL_FLOAT	4
STR_CALIB_TIME	STR_CALIB_TIME	SQL_TIME	12
STR_RESID_DOSE	STR_RESID_DOSE	SQL_FLOAT	4
STR_RESID_TIME	STR_RESID_TIME	SQL_TIME	12
STR_INJ_TIME	STR_INJ_TIME	SQL_TIME	12
STR_IMG_TIME	STR_IMG_TIME	SQL_TIME	12
STR_IMG_DUR	STR_IMG_DUR	SQL_FLOAT	4
STR_INJECT_IMAGE_DELAY	STR_INJECT_IMAGE_DELAY	SQL_FLOAT	4

4DM VARIABLE NAME	DEFAULT FIELD IN TABLE	DATA TYPE	SIZE
REST_RADIOPHARMACEUTICAL	REST_RADIOPHARMACEUTICAL	SQL_CHAR	18
REST_DOSE	REST_DOSE	SQL_FLOAT	4
RST_CALIB_TIME	RST_CALIB_TIME	SQL_TIME	12
RST_RESID_DOSE	RST_RESID_DOSE	SQL_FLOAT	4
RST_RESID_TIME	RST_RESID_TIME	SQL_TIME	12
RST_INJ_TIME	RST_INJ_TIME	SQL_TIME	12
RST_IMG_TIME	RST_IMG_TIME	SQL_TIME	12
RST_IMG_DUR	RST_IMG_DUR	SQL_FLOAT	4
RST_INJECT_IMAGE_DELAY	RST_INJECT_IMAGE_DELAY	SQL_FLOAT	4

STR_EF	STR_EF	SQL_FLOAT	4
STR_EDV	STR_EDV	SQL_FLOAT	4
STR_ESV	STR_ESV	SQL_FLOAT	4
STR_MYOMASS	STR_MYOMASS	SQL_FLOAT	4
STR_TID	STR_TID	SQL_FLOAT	4
STR_CO	STR_CO	SQL_FLOAT	4
STR_EDVI	STR_EDVI	SQL_FLOAT	4
STR_ESVI	STR_ESVI	SQL_FLOAT	4
STR_CI	STR_CI	SQL_FLOAT	4
STR_PEAK_FILLING_RATE	STR_PEAK_FILLING_RATE	SQL_FLOAT	4
STR_PEAK_EJECTION_RATE	STR_PEAK_EJECTION_RATE	SQL_FLOAT	4
STR_ONETHIRDMFR	STR_1_3MFR	SQL_FLOAT	4
STR_T_PEAK_FILLING_RATE	STR_T_PEAK_FILLING_RATE	SQL_FLOAT	4
STR_T_PEAK_EJECTION_RATE	STR_T_PEAK_EJECTION_RATE	SQL_FLOAT	4
STR_IMAGING_HR	STR_IMAGING_HR	SQL_INTEGER	2

RST_EF	RST_EF	SQL_FLOAT	4
RST_EDV	RST_EDV	SQL_FLOAT	4
RST_ESV	RST_ESV	SQL_FLOAT	4
RST_MYOMASS	RST_MYOMASS	SQL_FLOAT	4
RST_CO	RST_CO	SQL_FLOAT	4
RST_EDVI	RST_EDVI	SQL_FLOAT	4
RST_ESVI	RST_ESVI	SQL_FLOAT	4
RST_CI	RST_CI	SQL_FLOAT	4
RST_PEAK_FILLING_RATE	RST_PEAK_FILLING_RATE	SQL_FLOAT	4
RST_PEAK_EJECTION_RATE	RST_PEAK_EJECTION_RATE	SQL_FLOAT	4
RST_ONETHIRDMFR	RST_1_3MFR	SQL_FLOAT	4
RST_T_PEAK_FILLING_RATE	RST_T_PEAK_FILLING_RATE	SQL_FLOAT	4
RST_T_PEAK_EJECTION_RATE	RST_T_PEAK_EJECTION_RATE	SQL_FLOAT	4
RST_IMAGING_HR	RST_IMAGING_HR	SQL_INTEGER	2

STR_LAD_EXT	STR_LAD_EXT	SQL_FLOAT	4
STR_LCX_EXT	STR_LCX_EXT	SQL_FLOAT	4
STR_RCA_EXT	STR_RCA_EXT	SQL_FLOAT	4

4DM VARIABLE NAME	DEFAULT FIELD IN TABLE	DATA TYPE	SIZE
STR_TOT_EXT	STR_TOT_EXT	SQL_FLOAT	4
RST_LAD_EXT	RST_LAD_EXT	SQL_FLOAT	4
RST_LCX_EXT	RST_LCX_EXT	SQL_FLOAT	4
RST_RCA_EXT	RST_RCA_EXT	SQL_FLOAT	4
RST_TOT_EXT	RST_TOT_EXT	SQL_FLOAT	4
REV_LAD_EXT	REV_LAD_EXT	SQL_FLOAT	4
REV_LCX_EXT	REV_LCX_EXT	SQL_FLOAT	4
REV_RCA_EXT	REV_RCA_EXT	SQL_FLOAT	4
REV_TOT_EXT	REV_TOT_EXT	SQL_FLOAT	4

SSS	SSS	SQL_INTEGER	2
SRS	SRS	SQL_INTEGER	2
SDS	SDS	SQL_INTEGER	2
STS_S	STS_S	SQL_INTEGER	2
STS_R	STS_R	SQL_INTEGER	2
SMS_S	SMS_S	SQL_INTEGER	2
SMS_R	SMS_R	SQL_INTEGER	2

PERFUSION_FINDINGS	PERFUSION_FINDINGS	SQL_CHAR	1024
MYOCARDIAL_PERFUSION	MYOCARDIAL_PERFUSION	SQL_CHAR	1024
VESSEL_DISEASE	VESSEL_DISEASE	SQL_CHAR	40
SCAN_SIGNIFICANCE	SCAN_SIGNIFICANCE	SQL_CHAR	80
EVENT_RISK_SUMMARY	EVENT_RISK_SUMMARY	SQL_CHAR	1024
TID_FINDINGS	TID_FINDINGS	SQL_CHAR	60

FUNCTIONAL_FINDINGS	FUNCTIONAL_FINDINGS	SQL_CHAR	1024
LV_GLOBAL_SYSTOLIC_FUNCTION	LV_GLOBAL_SYSTOLIC_FUNCTION	SQL_CHAR	40
LV_REGIONAL_WALL_MOTION	LV_REGIONAL_WALL_MOTION	SQL_CHAR	1024
LV_REGIONAL_WALL_MOTION_ZONES	LV_REGIONAL_WALL_MOTION_ZONES	SQL_INTEGER	2
LV_VOLUME	LV_VOLUME	SQL_CHAR	40
LV_FINDINGS_SUMMARY	LV_FINDINGS_SUMMARY	SQL_CHAR	1024
RV_PERFUSION	RV_PERFUSION	SQL_CHAR	1024
RV_REGIONAL_WALL_MOTION	RV_REGIONAL_WALL_MOTION	SQL_CHAR	1024
RV_REGIONAL_WALL_MOTION_ZONES	RV_REGIONAL_WALL_MOTION_ZONES	SQL_INTEGER	2
RV_VOLUME	RV_VOLUME	SQL_CHAR	40
RV_FINDINGS_SUMMARY	RV_FINDINGS_SUMMARY	SQL_CHAR	1024

EXTRA_CARD_ACTIVITY	EXTRA_CARD_ACTIVITY	SQL_CHAR	256
OVERALL_STUDY_QUALITY	OVERALL_STUDY_QUALITY	SQL_CHAR	256
SUMMARY	SUMMARY	SQL_CHAR	4096

Study Table Field Names

4DM VARIABLE NAME	DEFAULT FIELD IN TABLE	DATA TYPE	SIZE
PAT_NAME	PAT_NAME*	SQL_CHAR	32
PAT_ID	PAT_ID*	SQL_CHAR	24
STUDY_DATE	STUDY_DATE*	SQL_DATE	12
STUDY_TIME	STUDY_TIME*	SQL_TIME	12
STUDY_IDTYPE	STUDY_IDTYPE*	SQL_INTEGER	4
STUDY_IDSTRING	STUDY_IDSTRING*	SQL_CHAR	32
PAT_AGE	PAT_AGE	SQL_INTEGER	2
PAT_SEX	PAT_SEX	SQL_CHAR	16
PAT_WGT	PAT_WGT	SQL_INTEGER	2
PAT_HGT	PAT_HGT	SQL_FLOAT	4
PAT_BSA	PAT_BSA	SQL_FLOAT	4

REP_PHYSICIAN	REP_PHYSICIAN	SQL_CHAR	32
DATA_STATE	DATA_STATE	SQL_CHAR	64
EXPORT_TIME	EXPORT_TIME	SQL_CHAR	24

IMAGING_PROTOCOL	IMAGING_PROTOCOL	SQL_CHAR	64
RADIOPHARMACEUTICAL	RADIOPHARMACEUTICAL	SQL_CHAR	18
CALIB_DOSE	CALIB_DOSE	SQL_FLOAT	4
CALIB_TIME	CALIB_TIME	SQL_TIME	12
RESID_DOSE	RESID_DOSE	SQL_FLOAT	4
RESID_TIME	RESID_TIME	SQL_TIME	12
INJECT_TIME	INJECT_TIME	SQL_TIME	12
IMAGE_DUR	IMAGE_DUR	SQL_INTEGER	2
IMAGE_TIME	IMAGE_TIME	SQL_TIME	12
INJECT_IMAGE_DELAY	INJECT_IMAGE_DELAY	SQL_FLOAT	4

EF	EF	SQL_FLOAT	4
EDV	EDV	SQL_FLOAT	4
EDVI	EDVI	SQL_FLOAT	4
ESV	ESV	SQL_FLOAT	4
ESVI	ESVI	SQL_FLOAT	4
MYOMASS	MYOMASS	SQL_FLOAT	4
TID	TID	SQL_FLOAT	4

*Field names with * are keyed fields in table.*

4DM VARIABLE NAME	DEFAULT FIELD IN TABLE	DATA TYPE	SIZE
CO	CO	SQL_FLOAT	4
CI	CI	SQL_FLOAT	4
PEAK_FILLING_RATE	PEAK_FILLING_RATE	SQL_FLOAT	4
PEAK_EMPTYING_RATE	PEAK_EMPTYING_RATE	SQL_FLOAT	4
ONETHIRDMFR	1_3MFR	SQL_FLOAT	4
T_PEAK_FILLING_RATE	T_PEAK_FILLING_RATE	SQL_FLOAT	4
T_PEAK_EMPTYING_RATE	T_PEAK_EMPTYING_RATE	SQL_FLOAT	4
IMAGING_HR	IMAGING_HR	SQL_INTEGER	2

LAD_EXT	LAD_EXT	SQL_FLOAT	4
LCX_EXT	LCX_EXT	SQL_FLOAT	4
RCA_EXT	RCA_EXT	SQL_FLOAT	4
TOT_EXT	TOT_EXT	SQL_FLOAT	4
REV_LAD_EXT	REV_LAD_EXT	SQL_FLOAT	4
REV_LCX_EXT	REV_LCX_EXT	SQL_FLOAT	4
REV_RCA_EXT	REV_RCA_EXT	SQL_FLOAT	4
REV_TOT_EXT	REV_TOT_EXT	SQL_FLOAT	4

REG_PERF_SCORE	REG_PERF_SCORE	SQL_CHAR	64
REG_WTHK_SCORE	REG_WTHK_SCORE	SQL_CHAR	64
REG_WMOT_SCORE	REG_WMOT_SCORE	SQL_CHAR	64
SUM_PERF_SCORE	SUM_PERF_SCORE	SQL_INTEGER	2
SUM_WTHK_SCORE	SUM_WTHK_SCORE	SQL_INTEGER	2
SUM_WMOT_SCORE	SUM_WMOT_SCORE	SQL_INTEGER	2